

### Easy Steps To Apply:

**1** Prepare site according to facility protocol. Be sure you have the proper size **AnchorDry** to cover site that you want to prevent from getting wet. The **AnchorDry** dressing should extend beyond the IV dressing.



**2** Apply **AnchorDry** to clean dry skin. Lotion may inhibit the adhesive qualities of **AnchorDry**. Before application, dog ear the adhesive backing at the 4 corners shown to the right.



**3** Remove top adhesive backing and adhere to skin.



**4** Remove one side of the adhesive backing and adhere to skin. Repeat process on opposite side.



**5** Remove the bottom adhesive backing and adhere to skin.



**6** Pinch together the gap and rotate back and forth to improve the seal as it adheres to the skin. The adhesive may be used to seal around any tubing.

**7** Smooth outer border using firm pressure to ensure a proper seal.



**8** After showering, carefully remove **AnchorDry** in the direction of hair growth and dispose of accordingly.



**AnchorDry Sizes:**  
5x5, 7x7, 9x9,  
10x12, 12x14 and 7x22



#### CAUTIONS & WARNINGS:

- ◆ Discontinue use if patient has an adverse reaction to or irritation from the product.
- ◆ **AnchorDry** should be routinely inspected during showering.
- ◆ Discard if packaging is opened or damaged.
- ◆ One time use.
- ◆ Dispose of accordingly after use.
- ◆ Latex-Free.