




iMedTechnology, Inc.

PICC Anchor

The **PICC ANCHOR** is for use with Peripherally Inserted Central Catheters (PICCs), this **IV Securement** offers simple, secure placement.

Features & Benefits

- Extremely easy to apply and remove.
- Universal size secures any size or type of PICC catheter wings (hub).
- Low profile design is easy to conceal. Maximizes patient comfort.
- Flexible material is soft, breathable, latex-free and conforms comfortably to the arm.
- Patient-friendly adhesive is gentle, yet stays in place for long wear times, reducing the number of unscheduled dressing changes.
- The Clear Window Flap allows the nurse to view the catheter wings (hub).
- Contoured shape works with "Bio-Patch" type devices allowing positioning close to the insertion site.
- Provides catheter stabilization as recommended by the 2011 INS Standards of Practice and the 2011 CDC Guidelines.
- Can be used on Adult and Pediatric patients.
- Sterile, individually packed in boxes of 50 pieces.

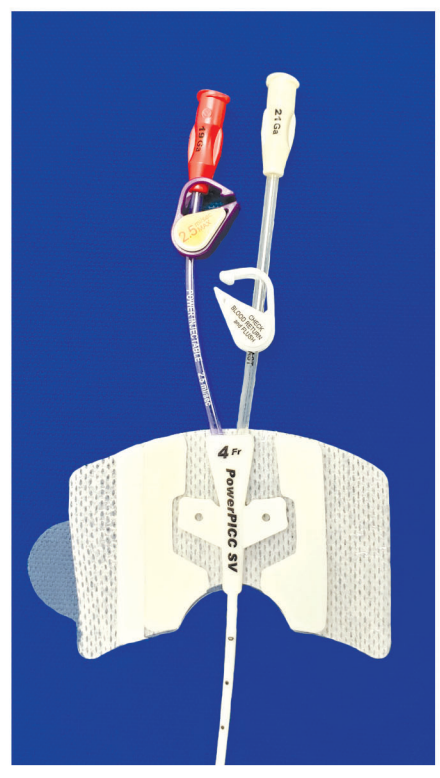

Instructions For Use

- 1 Prepare the skin in accordance with established practice. Hair removal may be required for better adhesion.
- 2 Open the top Clear Window Flap and place the PICC Anchor under the catheter wings (hub). Remove the center peel paper, center the catheter wings (hub) in the adhesive area and press gently into the adhesive.
- 3 Close the Clear Window Flap on the catheter wings (hub) which can be repositioned, allowing the catheter to be inspected and adjusted.
- 4 Remove the bottom peel paper from one side while holding the PICC Anchor in place, remove the other peel paper, press in place on the skin.

iMed Technology, 2515 Tarpley Road, Suite 104, Carrollton, TX 75006, PH: 972-416-8800

www.imedtechnology.com

iMedTechnology, Inc.

