



SUCTION CANISTER TUBING FOR USE ON iMED SUCTION CANISTERS

REF iST

QTY - 1 EA

INSTRUCTIONS FOR TUBING USE

USING THE iMED 600ML OR 1000ML DRAINAGE BOTTLE WITH SEPARATE DRAIN LINE

Follow these step-by-step directions.

- 1 Pick up the bag that contains the iMed 600ml or 1,000ml drainage bottle without drain line. Look at the green vacuum indicator on the top of the bottle. This indicator tells you if there is a vacuum in the bottle. This indicator should be down, not up. If the indicator is up discard and use another bottle. Open the package and remove the bottle.
- 2 Open the sterile peel package and take out the iMed Suction Canister drainage tubing, model iST.
- 3 Attach the drainage tubing connector to the female luer connection that is attached to the bottle and twist firmly to close. Close the white slide clamp on the tubing.
- 4 Attach a cannula to the male luer connector at the end of the tubing set. Insert cannula to patient according to institution protocol.
- 5 Slide the white slide clamp on the green tubing until it no longer pinches the green tubing closed. Release the white pinch clamp on the drain line slowly to begin draining fluid. The fluid will flow into the vacuum bottle.
- 6 After the drainage procedure; close the white tubing slide clamp. Remove cannula from patient.
- 7 Dispose the iMed 600ml or 1,000ml drainage bottle and drain line according to facility policy and procedures.



Read instructions prior to use

STERILE EO

Sterilized with ETO



Single use

Sterility guaranteed unless package is opened or damaged.



Do not re-sterilize

Not made with Natural Rubber Latex



Store in cool, dry area

LOT



Do not use if package is opened or damaged

