



DRAINAGE BAGS FOR USE ON RENOVA™ PUMP

REF iPB2500

QTY - 2 EA

INSTRUCTIONS FOR BAG USE

- 1 CAUTION:** Be sure pump is turned off to load the tubing set. Lift the pump front door until fully open. Set the middle of the thicker silicone tubing across the pump-head, (Rotor wheel). The left side of the tubing goes to the patient. The right side goes to the drainage bags.
- 2** Close the pump door over the tubing/pump-head assembly.
- 3** Load the “T” shaped section of the tubing in the recessed area at the top of the pump.
- 4** Load drainage bags on the 2 posts on the sides of the pump and attach to the “T” shaped tubing assembly. Tighten the tubing connections to prevent leakage or aspiration of air.
- 5** Open or close the 2 stopcocks to direct the flow of fluid to the left or right drainage bag.
- 6** When drainage bag is full (or procedure is complete), turn pump off, turn stopcocks to “off” position, disconnect tubing from bag and attach blue cap tightly to the tubing luer lock on bag.



Read instructions prior to use



Single use



Do not re-sterilize



Store in cool, dry area



Do not use if package is opened or damaged

STERILE EO

Sterilized with ETO

Sterility guaranteed unless package is opened or damaged.

Not made with Natural Rubber Latex

LOT

