



# DRAINAGE BAG KIT FOR USE ON RENOVA™ PUMP

**REF** iPBK2500

## INSTRUCTIONS FOR BAG USE

- 1 **CAUTION:** Be sure pump is turned off to load the tubing set. Lift the pump front door until fully open. Set the middle of the thicker silicone tubing across the pump-head, (Rotor wheel). The left side of the tubing goes to the patient. The right side goes to the drainage bags.
- 2 Close the pump door over the tubing/pump-head assembly.
- 3 Load the “T” shaped section of the tubing in the recessed area at the top of the pump.
- 4 Load drainage bags on the 2 posts on the sides of the pump and attach to the “T” shaped tubing assembly. Tighten the tubing connections to prevent leakage or aspiration of air.
- 5 Open or close the 2 stopcocks to direct the flow of fluid to the left or right drainage bag.
- 6 When drainage bag is full (or procedure is complete), turn pump off, turn stopcocks to “off” position, disconnect tubing from bag and attach blue cap tightly to the tubing luer lock on bag.

## INSTRUCTIONS FOR DRESSING THE PUNCTURE SITE

**WHEN DRAINAGE IS COMPLETED AND CANNULA IS REMOVED:**

- 1 Clean and dress the puncture site according to hospital protocol.
- 2 Use one or two of the adhesive bandages to cover the puncture site after cleaning.
- 3 Fold one or more 4x4 gauze pads and cover the adhesive bandages.
- 4 Cover the adhesive bandages and gauze pads with the transparent dressing for up to 24 hours.
- 5 Remove transparent dressing, gauze pads and adhesive bandages after the puncture site is closed.
- 6 Redress if necessary.

## KIT CONTENTS

- (2) Waste Bags
- (1) Tubing Set
- (2) Adhesive Bandages
- (2) 4x4 Gauze Pads
- (1) Transparent Dressing



Read instructions prior to use



Single use



Do not re-sterilize



Store in cool, dry area



Do not use if package is opened or damaged

**STERILE EO**

Sterilized with ETO

Sterility guaranteed unless package is opened or damaged.

Not made with Natural Rubber Latex

**LOT**



**DISPOSE OF ALL WASTE PER HOSPITAL PROTOCOL**

iMED Technology, 2515 Tarpley Road, Suite 104, Carrollton, TX 75006, PH: 972-416-8800

[www.imedtechnology.com](http://www.imedtechnology.com)